Materials to be exposed to the body
Needle Tube: Stainless Steel
Needle Hub: Polycarbonate, Silicone Rubber (space filler)

Intended Use, Indication

Single-use sterile needle intended to be used with a syringe for medical purposes to administer parenteral solution into or withdraw fluid from the body. Appropriate gauge size will be chosen by medical professionals taking purpose of injection, applied area, patient condition and other necessary elements into consideration.

Contraindication

The device is for single use only. Do not resterilize or reuse. Reuse of the device could increase the risk of infection.

Description

ZELOSTAT Needles are composed of a needle tube and a needle hub. The hub has female luer lock connector to fit to luer lock syringes. The needle tubes vary in length (4mm to 25mm) and diameter (0.16mm to 0.50mm) depending on intended use.

Directions for Use

- 1) Remove the End Sheath and attach syringe to the Needle Hub.
- 2) Pull the Needle Sheath straight off the device.
- 3) If applicable, draw up injection fluids in accordance with your institution's protocol. Holding the syringe with the needle pointing up, check the syringe for bubbles. If there are bubbles, gently tap the syringe with your finger to remove any visible bubbles and expel them from the syringe.
- 4) Insert the needle and administer injection fluids or withdraw body fluids in accordance with your institution's protocol.
- 5) Dispose of the device promptly as infectious waste in accordance with your institution's protocol.

Important Cautions


- ASTI Corporation accepts no responsibility for accidents or defectives caused by misuse or abnormal use.
- Do not use the device if package is opened or damaged.
- Do not use the device if any sign of product damage is visible.
- Use the device immediately after opening the unit package.
- Do not touch the luer lock connector directly before use to avoid contamination.
- Do not touch the needle tubes directly while and after needle sheath is removed to avoid needle stick injury or infection.
- Ensure the device fits syringes firmly and confirm there is no defect in the fitting, like leakage.
- Make sure to use luer lock syringe. Using luer slip syringe may cause detachment of the device during injection.
- Do not excessively rotate while attaching to syringe.

Note

If there is any serious incident occurred in relation to the ZELOSTAT Needles, do report to ASTI and competent authority which your institution belongs to.

Storage Method and Expiration Date

- Keep the product in clean area, and avoid exposure to water, high temperature/humidity and direct sunlight.
- Expiration date is indicated on the individual package.

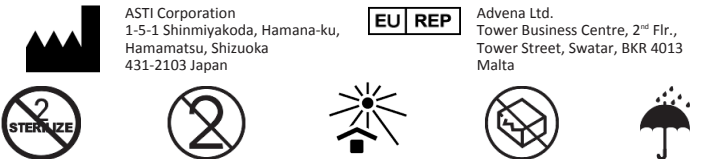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